**CONECTANDO
FRONTERAS**

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VIDAS : Seguimos Avanzando!

Christian Fraizer Alvarado

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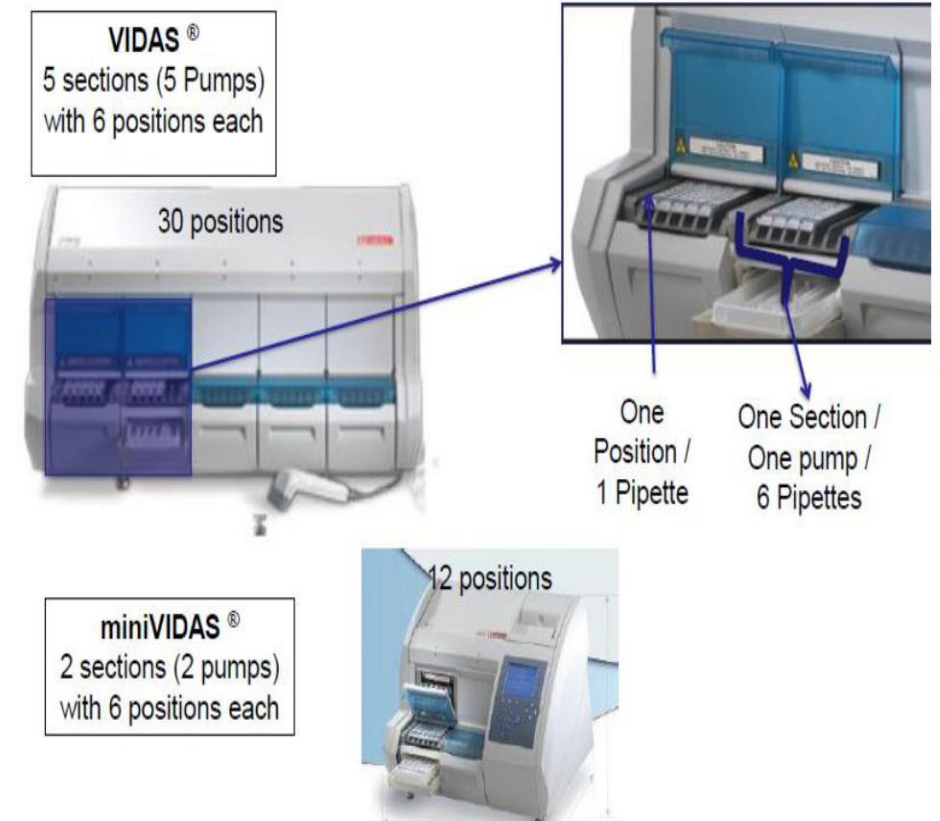
Figure 1 : mini VIDAS and VIDAS – the sections are represented in blue

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**CONECTANDO
FRONTERAS**

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NOTIFICACIONES VIDAS QCV

CORPORATE REGULATORY & QUALITY COMPLIANCE

BIOMERIEUX

Customers come first

FIELD SAFETY CORRECTIVE ACTION

Number: 3749 Action: Correction Unit Name: Immunology
Product Name: mini VIDAS and VIDAS clinical in combination with Quality Control VIDAS® (QCV)
Issue Date: 04-Jan-2018 Manufacturing Site: La Balme [Required Actions](#)

Number: 4691-1
Title: VIDAS® PC and MINI VIDAS® - QCV® Alert reminder and new Service Manuals
This MAR applies to: BOTH CLINICAL AND INDUSTRY

Catalog Profile: C09, C11

System Name: VIDAS®, MINI VIDAS®

Lot/Serial: N/A

Expiry date: N/A

Required Actions: [See section B](#) – Deadline: 31-OCT-2020

Issue Date: March 20, 2020

BIOMERIEUX

As described in the Instructions for Use of QCV tests: "Quality Control VIDAS (QCV) is used to detect abnormal operation of pipette mechanisms which may affect the results of biological tests. It is also intended for checking that the optical system is capable of measuring high fluorescence levels.

The QCV test must be run at least once a month in each position of the VIDAS and mini VIDAS instruments, or any time that one of the above-mentioned instrument problems is suspected. It is essential to perform the VIDAS QCV test on all the positions to ensure that the whole instrument has been checked."

1) Required customer actions:

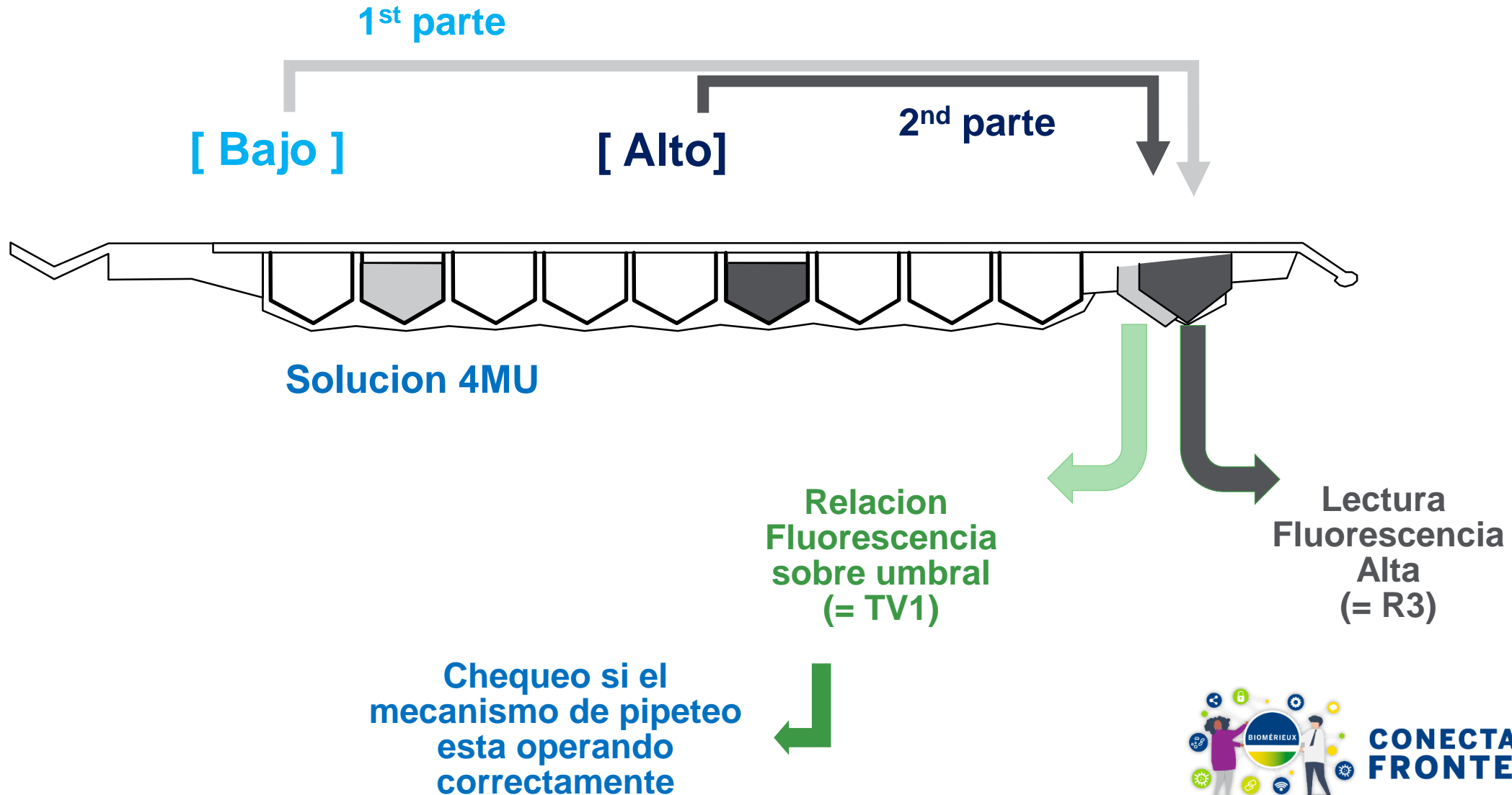
- Quality Control VIDAS® (QCV®) test:
 - Clinical: perform the QCV® on a weekly basis, to reduce the period of potential retrospective analyses needed;
 - Industry: perform the QCV® on a monthly basis and, in addition, possibly activate the VRC.
- Visual inspection of the SPR® after each run (including patient sample test and QCV® test) to ensure that the dot on the SPR® is still in place.

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WAS® Flow cytometry

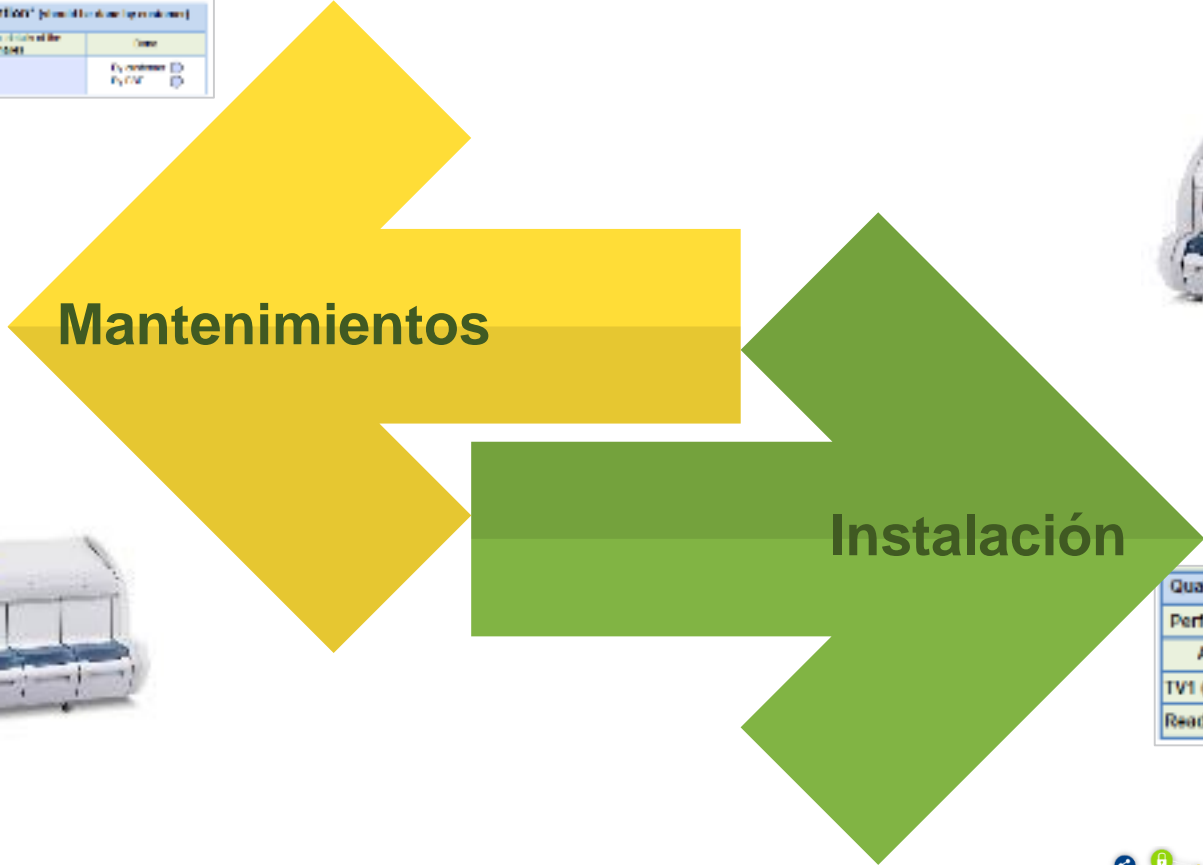
Qualification before intervention* (please check regularly)

Test	Control	Frequency/Interval	Date
QCV	TV1 TV2	30/30/30	By customer By QCV

MIRAS® Flow cytometry

Qualification before intervention* (please check regularly)

Test	Control	Frequency/Interval	Date
QCV	TV1 TV2		By customer By QCV

Quality the pipetting with QCV		
Perform qualification with QCV (3278)		<input type="checkbox"/>
Acceptance criteria	Comments	
TV1 (read2/read1) ≥ TV1kit		<input type="checkbox"/>
Reading 3 ≥ 4100 RFU		<input type="checkbox"/>

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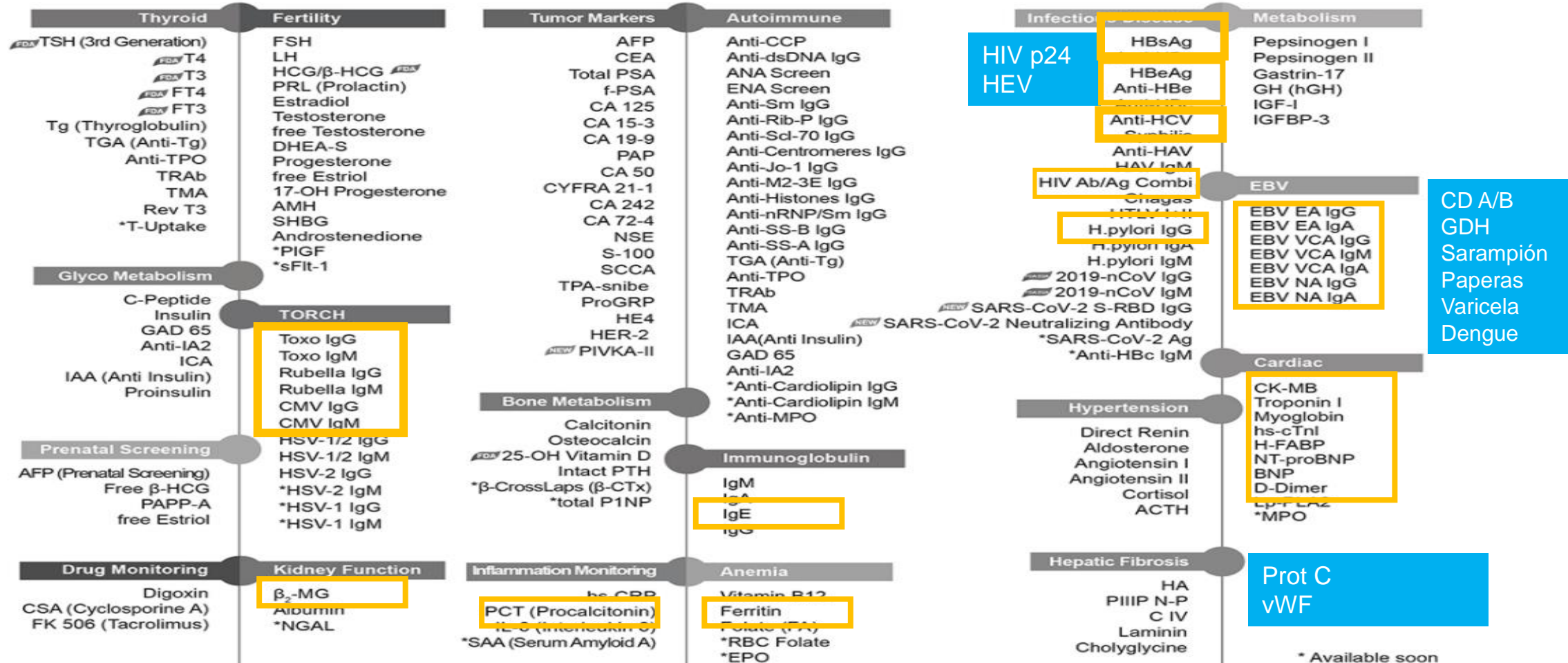
**CONECTANDO
FRONTERAS**

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Broadest CLIA Test Menu with 166 Assays

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Platform

Platform	MAGLUMI® series
Type	IA
Throughput	MAGLUMI 2000 Mid (180 tests/hr)
CLIA (ChemiLuminescent ImmunoAssay)	
Automated sensitive assay	

Sample matrix	Whole blood, Serum, plasma
Sample volume	20 µL
Test time	15 minutes
Measuring range	0.13 – 100 µg/L
Limit of quantification (LoQ)	0.13 µg/L
Limit of detection (LoD)	<i>No information available</i>
Precision (total 20 days)	< 8.9 %

Test

Test kit	Calibrators	Controls	Calibration stability
<i>No information available</i>	2 calibrators	2 controls (included)	2 weeks

Labeling	
Progression to severe sepsis/septic shock	
Prognosis	
Antibiotic stop in sepsis	
LRTI start/stop antibiotics	

Compare to VIDAS®
B·R·A·H·M·S PCT™

Compare to all competitors

Take away

STRENGTHS

- Low sample volume
- 2 calibrations
- 2 controls

WEAKNESSES

- **CE Approved but for not any Claim**
- **No validated publications**
- **No FDA Clearance**
- Measuring range
- Low specificity

KEEP IN MIND

- Snibes's Maglumi PCT declared unsafe by the Medical Devices Scientific Commission of Turkey in 2015
- No publication to prove its efficacy and safety

MAGLUMI® D-DIMER

Not FDA cleared for VTE exclusion

Platform

Platform	Maglumi® series
Type	IA
Throughput	Maglumi® 600 : 180T/hr
	Maglumi® 2000 : 180T/hr
	Maglumi® 4000 :240T/hr
CLIA (Chemiluminescence Immunoassay)	
Full automation	Yes

Test

Sample matrix	Serum or plasma
Sample volume	20µL
Test time	17 mins
Measuring range	Not available*
Limit of detection (LoD)	Not available*
Cut-off	0,5µg/mL**
CV at CO	Not available*

Test kit	Calibrators	Controls	Calibration stability
100	2 point included	Included	Every week

Intended use	D-Dimer quantification
PE Sensitivity	Not available*
PE NPV	Not available*
DVT Sensitivity	Not available*
DVT NPV	Not available*

- Compare to VIDAS® D-Dimer Exclusion
- Compare to all competitors

Take away

STRENGTHS

- Throughput
- Full automation
- All inclusive kit
- Sample volume

WEAKNESSES

- Not FDA cleared for VTE exclusion
- Intended use is not VTE exclusion
- Manual preparation of the reagent in 30 mins
- Calibration every week
- Quality controls must be run daily
- 4 weeks on board stability
- Reaction modules must be purchased separately

KEEP IN MIND

- Limited scientific evidence. Very limited information in the PI
- Quality issues – batch to batch variations.
- Not FDA cleared for VTE Exclusion
- Intended use is not VTE exclusion
- No age adjusted cut-offs validated
- Not validated for HERDOO 2 claim†
- Additional costs with consumables

MAGLUMI TROPONIN I CLIA

Platform

Platform	Maglumi® family
Type	IA
Throughput	800 & 2000: 180T/HR
CLIA	
Full automation	Yes

Sample matrix	Plasma or serum
Sample volume	100µL
Test time	17 minutes
Measuring range	0.1 – 50ng/mL
Limit of detection (LoD)	0.10ng/ml
Limit of quantification	Not available
99 th percentile	0.2ng/ml.
CV at 99th percentile	Not available
NEP	Not available

Test

Test kit	On board stability	Calibrators	Controls	Calibration frequency	Controls frequency
100	4 weeks	2 included	Not provided	Every week	Every 24hrs

Intended use / claim	Trop I quantification
Validated algorithms	No
Sex specific CO	No
Rule in CO	No

- Compare to VIDAS® HS - Trop I
- Compare to all competitors

Take away

STRENGTHS

- Fully automated systems

WEAKNESSES

- Controls not provided by the company
- Quality controls daily
- Calibration weekly

KEEP IN MIND

- Additional costs with controls
- Conventional troponin I test, not achieving the High Sensitivity requirements
- No rule and no sex specific cut offs with this test
- Limited information shared in the package insert

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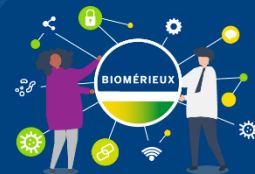
HIV p24
HEV

Thyroid

FT3
FT4
T3
T4
TSH
Anti-TG
Anti-TPO
Tg
rT3
TRAB*

Bone

Metabolic
Intact PTH
Calcitonin
Vitamin D total

Infectious Disease

HIV Combo
HBsAg
Anti-HBs
HBeAg
Anti-HBe
Anti-HBc
Anti-HCV
Anti-TP (Syphilis)
HBeAg quantitative*
Anti-HBc IgM*

Cardiac

CK-MB
Myoglobin
Troponin I
BNP

Fertility

Total β -HCG
FSH
LH
Prolactin
E2
E3
TESTO
PROG
Free testosterone*
17-OH PROG*
SHBG*

AMH*

Diabetes

Insulin
C-peptide

Tumor marker

CEA
AFP
CA125
CA15-3
CA19-9
FPSA
TPSA
NSE
CYFRA 21-1
CA72-4
PG I
PG II
SCCA
HE4
ProGRP
CA50
CA242

Adrenal Function

DHEA-S
Cortisol
ACTH

Hypertension

Angiotension I*
Angiotension II*
Renin*

Liver Fibrosis

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Growth Hormone

hGH*
IGF-1*

mindray

ToRCH

Toxo IgG*
Toxo IgM*
Rubella IgG*
Rubella IgM*
CMV IgG*
CMV IgM*
HSV-1/2 IgG*
HSV-1/2 IgM*
HSV-1 IgG*
HSV-2 IgG*

CD A/B
GDH
Sarampión
Paperas
Varicela
Dengue

Anemia

Ferritin
Vitamin B12
Folate
RBC Folate

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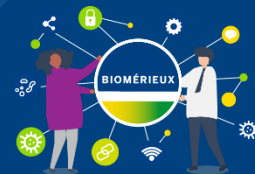
Not FDA cleared for VTE exclusion

Platform

Platform	AIA Series
Type	IA
Throughput	AIA 900 : 90T/hr AIA 2000 : 200 T/hr
Two-site immunoenzymatic assay	
Full automation	Yes

Test

Sample matrix	Citrated plasma
Sample volume	10µL
Test time	20 mins
Measuring range	0.02 – 16 µg/mL DDU
Limit of detection (LoD)	0.0132 µg/mL DDU
Cut-off	0,5µg/mL
CV at CO	4.8% at 0.274 µg/mL DDU

Test kit	Calibrators	Controls	Calibration stability
100	1 set Not included	1 set Not included	90 days

Intended use	Aid to diagnostic
PE Sensitivity	Not available*
PE NPV	Not available*
DVT Sensitivity	Not available*
DVT NPV	Not available*

- Compare to VIDAS® D-Dimer Exclusion
- Compare to all competitors

Take away

STRENGTHS

- LOD, measuring range
- Full automation
- Continuous loading
- Calibration stability
- Sample volume

WEAKNESSES

- Not FDA cleared for VTE exclusion
- Intended use is not VTE exclusion
- Quality controls must be run daily
- Calibrators and controls are not included in the kit
- Material required but not provided : diluents, pipettes, cups, preloaded pipettes, wash solution, substrate, sample treatment not included

KEEP IN MIND

- Not FDA cleared for VTE exclusion
- Intended use is not VTE exclusion
- No Age adjusted cut-offs validated
- Not validated for HERDOO 2 claim†
- Additional costs with controls, calibrators and consumables
- Limited scientific evidence

CL AIA D-DIMER PACK

Not FDA cleared for VTE exclusion

Platform

Platform	CL AIA Series
Type	IA
Throughput	CL 1200 : 120T/hr
	CL 2400 : 240 T/hr
CLEIA (Chemiluminescence immunoassay)	
Full automation	Yes

Sample matrix	Citrated plasma
Sample volume	10µL
Test time	15 mins
Measuring range	0.02 – 16 µg/mL DDU
Limit of detection (LoD)	0.0016 µg/mL DDU
Cut-off	0,5µg/mL
CV at CO	2.9% at 0.665 µg/mL

Test

Test kit	Calibrators	Controls	Calibration stability
96	1 set Not included	1 set Not included	90 days

Intended use	Aid to diagnostic
PE Sensitivity	Not available*
PE NPV	Not available*
DVT Sensitivity	Not available*
DVT NPV	Not available*

- Compare to VIDAS® D-Dimer Exclusion
- Compare to all competitors

Take away

STRENGTHS

- LOD, measuring range
- Full automation
- Continuous loading
- Calibration stability
- Sample volume

WEAKNESSES

- Not FDA cleared for VTE exclusion
- Intended use is not VTE exclusion
- Quality controls must be run daily
- Calibrators and controls are not included in the kit
- Material required but not provided : diluents, pipettes, cups, preloaded pipettes, wash solution, substrate, sample treatment not included

KEEP IN MIND

- Not FDA cleared for VTE exclusion
- Intended use is not VTE exclusion
- No Age adjusted cut-offs validated
- Not validated for HERDOO 2 claim†
- Additional costs with controls, calibrators and consumables
- Limited scientific evidence

ST AIA PACK CTNI 3RD GEN

Platform

Platform	AIA family
Type	IA
Throughput	600: 60T/HR
	900: 90T/HR
	1800: 180T/HR
Fluorometric Enzyme Immunoassay (FEIA)	
Full automation	Yes

Test

Sample matrix	EDTA or heparinized plasma/serum
Sample volume	50µL
Test time	18 minutes
Measuring range	0.02ng/mL-50ng/mL
Limit of detection (LoD)	0.008ng/mL
Limit of quantification	0.035ng/mL
99 th percentile	0.04ng/mL
CV at 99th percentile	3.1% at 0.436ng/mL
NPV	Not mentioned in the PI

Test kit	On board stability	Calibrators	Controls	Calibration frequency	Controls frequency
5x20	10 days	1 set not included	Not provided by tosoh	90 days	Once per day

Intended use / claim	AMI aid diagnosis
Validated algorithms	No
Sex specific CO	No
Rule in CO	No

- Compare to VIDAS[®] HS - TroP I
- Compare to all competitors

Take away

STRENGTHS

- Fully automated systems
- Calibration stability
- Multiple platforms adapted to different volumes

WEAKNESSES

- Limited on board stability
- Calibrators are not included in kit
- Controls are not provided by the company
- Quality controls every day
- Substrate reagents must be kept at room temperature before use

KEEP IN MIND

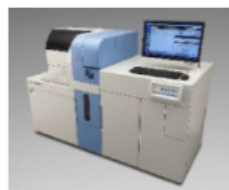
- Additional costs with controls and calibrators
- Conventional troponin I test, not achieving the High Sensitivity requirements
- No sex specific and no rule in cut offs available with this test
- No information regarding clinical performances in the PI

CL AIA-PACK® CTNI TEST CUP

Platform

Platform	CL AIA® analyzers
Type	IA
Throughput	CL1200: 120T/HR
	CL 2400: 240T/HR
CLEIA (Chemiluminescence enzyme immunoassay)	
Full automation	Yes

Sample matrix	EDTA or heparinized plasma/serum
Sample volume	50µL
Test time	18 mins
Measuring range	10 – 50 000pg/mL
Limit of detection (LoD)	1.61 pg/mL
Limit of quantification	Not mentioned in the PI
99 th percentile	31pg/mL
CV at 99 th percentile	4.7% at 129pg/mL
NPV	Not mentioned in the PI

Test

Test kit	On board stability	Calibrators	Controls	Calibration frequency	Controls frequency
3x32	30 days	Not included	Not provided by tosoh	90 days	Once per day

Intended use / claim	AMI aid diagnosis
Validated algorithms	No
Sex specific CO	No
Rule in CO	No

- Compare to VIDAS® HS - Tropon I
- Compare to all competitors

Take away

STRENGTHS

- Fully automated systems
- Calibration stability

WEAKNESSES

- Limited on board stability
- Calibrators are not included
- Controls not provided by the company
- Quality controls every day

KEEP IN MIND

- Additional costs with controls, calibrators and consumables
- Conventional troponin I test, not achieving the High Sensitivity requirements
- No sex specific and no rule in cut offs available with this test
- Not suitable for emergency / small volume
- No information regarding clinical performances in the PI

Immunoassays

- ✓ **Middle Segment** The **CL-6000i** has a high throughput with speeds of up to 480T/H, as well as greater expandability, which allows it to be integrated with the **SAL 9000**, the **BS-2000** and the new sample processing unit.

For higher immunology test speeds, two CL-6000i machines can be integrated to provide a throughput of almost up to 1000T/H.

- ✓ **Small segment** The **CL-900i** is a compact, fully-automated **CLIA analyzer**.

With dimensions of just 86cm length, 72cm width, 56cm height, specially designed for small volume labs.

Mindray has been promoting its chemiluminescence immunoassay business since 2008, when the company started the new business, many series of diagnosis machines have been launched, such as CL-2000i (2013), CL-1000i (2014) and the flagship product CL-6000i (2018).

vet business We expect Mindray to leverage on its rich experience in developing hospital medical equipment to quickly ramp up its veterinary equipment business before 2023. Vet solutions can be separated into two parts – veterinary monitors and veterinary ultrasound machines.

MAGLUMI

Immunoassays

Immunoassays Competition Immunoassays SNIBE communicates about the new Maglumi X3 a middle segment new strategy and the capacity to have synergies effects for all the portfolio instrument.

Throughput 200 tests/hr, compatible with all Maglumi reagents (166 parameters), RFID reader

Latest parameters launched TSH (3rd generation), T4, T3, FT4, FT3, HCG /B-HC, MPO, anti-MPO, 25-OH vitamin D, HSV 1 IgG

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